

# INSTRUCTIONS FOR USE

## EAGLE SLING - AVIATION USE ONLY HC26-0201 RANGE

Several different styles of slings are available for use with Haycomp brand patient lifters with cradle frames. All attach to the lifters in the same way. All slings are designed for maximum comfort, support and safety, for both user and carers. All Haycomp brand slings are made in Australia, using the highest quality available. All padded areas are of a closed cell, water repellent and chemically inert foam. Thus ensuring the padding will not harbour bacteria and / or moisture and will not be affected by bodily secretions or detergents. The slings are fully washable and durable, following the below instructions.

**All Slings are tested to International Standards ISO/DIS 10535 with a Safe Working Load S.W.L of 240Kg**

**IMPORTANT INFORMATION - DO NOT USE SLING IF LABEL IS NOT LEGIBLE OR MISSING!  
ALL SLINGS MUST BE REPLACED AFTER 5 YEARS**

### INSTRUCTIONS FROM SITTING POSITION - wheelchair to Aircraft + Aircraft to Wheelchair

1. Partially lean the client forward, so the sling can be slid down the back
2. Position the centre of the sling at clients back
3. Adjust and fasten firmly chest strap and leg strap
4. Position EAGLE lifter in front of passenger, so cradle is directly over the centre of gravity of passenger, ready to accept the sling attachments and lock the rear castor brakes
5. Lower the arm so the sling clips can be attached, attaching the shoulder clips first
6. Tilt the cradle down and attach the two leg clips
7. Ensure all 4 clips are securely engaged
8. Raise passenger (whilst holding the cradle handle) up slightly, so the sling just takes passenger's weight, but not fully off seat. Check that the sling is flat against the body and that your passenger is comfortable and safe.
9. The passenger may now be transferred to aircraft/wheelchair.



WASHING INSTRUCTIONS
ALWAYS USE WASHING BAG
CLOSE VELCRO + BUCKLES
MACHINE WASH MAX 70°
MILD DETERGENT
DO NOT BLEACH
TUMBLE DRY MAX 70°
DO NOT DRY CLEAN

SIZE	COLOUR CODES
XSML	WHITE
SMALL	GREEN
MEDIUM	YELLOW
LARGE	RED
XLGE	BLACK

ADDRESS
FOR REPAIRS, AUDITS AND SERVICING - <b>SHIP TO:</b>
HAYCOMP PTY LTD UNIT 1+2, 21 STUD RD BAYSWATER, VIC 3153

### **\*\* Please refer to your lifter's user manual for further instruction \*\***

- \* Always use slings with label and care instructions facing outwards, so correct side of sling is used
- \* Always store slings and accessories with velcro straps closed to prevent catching and piling of material
- \* Always position centre of sling at passenger's back centre
- \* Sling sizing is IMPORTANT to get right, if in doubt start with SMALL size first
- \* Passenger must NOT be left in sling unattended
- \* Sling fitting must be assessed by appropriately qualified personnel
- \* This sling can only be used on cradle frame "EAGLE LIFTER" attachments
- \* Regularly check the sling stitching and clips, if repairs are required - DO NOT USE and contact supplier immediately
- \* Always check S.W.L of sling and lifter. Use lesser S.W.L of the two as your maximum

OTHER HAYCOMP PRODUCTS FOR AVIATION
* Manual Handling Slings + Transfer Boards
* Ramps for Airbridges
* Chest straps, head straps, leg straps, JONY Belts
* Spare parts for your lifter

